## **Sporicidin<sup>®</sup> Bulletin No. 403**

### **Special Instructions for Cleaning and Decontamination of Surfaces/Objects Soiled with Blood/Bodily Fluids Against HIV (Human Immunodeficiency Virus)**

#### **Sporicidin<sup>®</sup> Disinfectant Spray (EPA Reg. Nos. 8383-4)**

**Personal Protection** Persons handling items soiled with blood or body fluids likely to be contaminated with HIV (Human Immunodeficiency Virus) must protect themselves by use of disposable latex gloves, gown, masks and eye covering.

**Decontamination of Contaminated Objects** Immediately after use, items contaminated with blood, body fluids, or other soils that may contain HIV (Human Immunodeficiency Virus) must be sprayed to thoroughly wet with Sporicidin<sup>®</sup> Disinfectant Solution for at least 1 minute before further processing.

**Cleaning Procedure** Following decontamination, objects and surfaces are moved to the cleaning and preparation areas where they are washed, rinsed and rough dried to remove all blood and body fluids before application of the disinfectant.

**Contact Time for Disinfection** Spray to thoroughly wet entire object or surface. HIV (Human Immunodeficiency Virus) is killed within 1-minute contact time. Note: Use a 10-minute contact time for other organisms. This product should not be used for disinfection of critical or semi-critical items.

**Disposal of Infectious Materials** Blood and other bodily fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

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